

Safe & Easy measurements for Veterinarians and Pet Owners!

DIGITAL HAND-HELD "POCKET" DOG URINE S.G. REFRACTOMETER **PAL-USG(DOG)**

Cat.No. 4510

DIGITAL HAND-HELD "POCKET" CAT URINE S.G. REFRACTOMETER **PAL-USG(CAT)**

Cat.No. 4511

The PAL-USG(DOG), PAL-USG(CAT) utilize the refractive index method to measure the specific gravity of urine.



Features

- Calibration with water only
- Measurement in 3 seconds
- Extremely water resistant (IP65)
- Light & Compact, 100g
- Equipped with a strap holder option
- Ergonomically designed for easy, one-handed operation
- External Light Interference (ELI)
- Convenient storage case
- Automatic Temperature Compensation (ATC)

Specification

	PAL-USG (DOG)	PAL-USG (CAT)
Measurement range	Urine S.G. 1.000~1.060	Urine S.G. 1.000~1.080
Resolution	Urine S.G. 0.001	
Measurement accuracy	Urine S.G. ±0.001	
Measurement temperature	10~35°C (ATC)	
Sample volume	0.3ml or more	
Measurement time	3 seconds	
Power supply	AAA alkaline battery × 2	
Battery life	Approx. 11,000 measurements (Using new alkaline batteries)	
International Protection Class	IP65 Water resistant	
Dimensions and weight	55(W)×31(D)×109(H)mm, 100g	

Measurement Method

1. Place the sample onto the prism surface.

2. Press the START key.

3. The measurement value will be displayed in 3 seconds.



※Quick & Easy Cleanup!



All ATAGO refractometers are designed and manufactured in Japan.



HACCP GMP GLP

ATAGO products comply with HACCP, GMP and GLP standards.

ATAGO CO., LTD.

Headquarters: 32-10, Honcho, Itabashi-ku, Tokyo 173-0001 Japan
TEL: 81-3-3964-6156 FAX: 81-3-3964-6137
overseas@atago.net http://www.atago.net/

ATAGO U.S.A., Inc.

12011 NE First Street, Bldg. C, Suite 110, Bellevue, WA 98005 U.S.A.
TEL: 1-425-637-2107 FAX: 1-425-637-2110
customerservice@atago-usa.com http://www.atago.net/

ATAGO INDIA Instruments Pvt. Ltd.

A-501, Mangal Aarambh Business Centre, Near Kora Kendra, Off. S.V. Road, Borivali (West), Mumbai-400 092 India
TEL: 91-22-2833-8038 / 8076 FAX: 91-22-2899-8164
customerservice@atago-india.com



※Specifications and appearance are subject to change without notice.



Sysmex Nederland B.V.
P.O. Box 251, 4870 AG Etten-Leur
Phone +31 (0)76 508 6000
Fax +31 (0)76 508 6086

Sysmex Belgium N.V.
Park Rozendal, Building A
Terhulpesteenweg 6a, 1560 Hoeilaart, Belgium
Phone +32 (0)2 769 7474 • Fax +32 (0)2 769 7499